

MATRx Titration Trays can be fit by a sleep dentist or a healthcare professional (non-dentist) that has been trained by a sleep dentist to perform tray fitting. All sleep labs performing MATRx studies should understand the training required for a healthcare professional (non-dentist) to be qualified to perform MATRx Titration Tray fitting.

The following should be considered if a healthcare professional (non-dentist) may be fitting MATRx Titration Trays:

1. A healthcare professional (non-dentist) must be trained, by a sleep dentist, to perform MATRx Titration Tray fitting.
2. The training provided by the sleep dentist should, at a minimum, include:
 - a. Review of the use of the MATRx Titration Tray Fitting Pre-Screening Questionnaire (Form Number: CS759)
 - b. Review of Zephyr documentation regarding the use of MATRx Titration Trays
3. The Pre-Screening Questionnaire must be completed by both the patient and trained healthcare professional (non-dentist) prior to MATRx Titration Tray fitting as follows:
 - a. The patient must complete and sign Part A of the questionnaire
 - b. The trained healthcare professional (non-dentist) must review the information provided by the patient in Part A and complete and sign Part B of the questionnaire
4. A patient that has any of the conditions specified in Part A and/or Part B of the Pre-Screening Questionnaire, should be referred to the sleep dentist for further assessment and MATRx Titration Tray fitting, if appropriate. In this case, the trained healthcare professional (non-dentist) is not qualified to perform the patient's tray fitting.